**HENROTEK®**

**HENROCUP®**

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**MARTS 2023**

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## **Important information**

Before using HENROCUP® read these instructions closely.

## **Intended use**

HENROCUP® is designed and intended for beverage and food only.

HENROCUP® has been developed by Danish occupational therapists to ensure a high-quality cup for people that experience loss of strength and coordination. HENROCUP® is not BPA Free and should not be used by children under the age of 3 years old.

HENROCUP® is designed with 2 wide handles to give a firm grip, the shape of the cup and handles ensures that you can drink without bending the neck. The bottom of the cup is wide to help it stand firm on the table and reduces spillage.

## **Storage**

Store dry, at room temperature and out of sunlight.

## **Instruction for use**

Content: 1 HENROCUP® can come with or without lid.

1. Unpack the cup and lid, dispose of the packing properly.
2. Before use wash the cup and lid.
3. The HENROCUP® are now ready for use.

## **Precautions**

After unpacking the cup always wash the HENROCUP® before use.

The cup intended use are for drinks.

Warm drinks can cause miscoloring and diminish the lifespan of the cup.

Do not heat the cup over a candle light or fire.

Be careful with hot fluid.

If used with lid, do make sure before use to check that the lid is put on correctly and is not loose.

## **Daily operation**

HENROCUP® can be washed in Dish Washer with up to 65° C degrees.

However, the cup can become dull and the quality deteriorates, so it is recommended to wash it by hand.

HENROCUP® can handle temperature range between -20° C to 95° C

## Warnings

Do not use any fluid in the HENROCUP® that is over 95° C in temperature.

**PLEASE DO NOTICE THAT SOME DRINKS MAY MISCOLOR THE INSIDE OF THE CUP.**

Dispose of the HENROCUP® if the surface does not feel smooth and changes to a milky-white color.

## Environmentally friendly disposal

Please dispose the cup in an environmentally friendly neutral way.

HENROCUP® are made of recyclable material which should be recycled. Please therefore dispose the old HENROCUP® via suitable collection systems.

**IF YOU HAVE ANY DOUBT OF WHEN TO DISPOSE THE HENROCUP®, YOU CAN SEND AN EMAIL TO [HENROTEK@HENROTEK.COM](mailto:HENROTEK@HENROTEK.COM)**

| <b>Technical datasheet HENROCUP®</b>  |  |
|---------------------------------------|--|
| <b>Material</b>                       | PC (Polycarbonate)   |
| <b>Weight</b>                         | 139,2 Gr   |
| <b>Height</b>                         | 9 cm   |
| <b>Width</b>                          | 15 cm  |
| <b>BPA</b>                            | HENROCUP® is not BPA Free and should not be used by children under the age of 3 years.   |
| <b>Marked</b>                         | CE   |
| <b>Color</b>                          | Clear with ml, Clear, Red, Green, Blue   |
| <b>Milliliters</b>                    | 230 ml   |
| <b>Dishwasher</b>                     | 65° C degrees  |
| <b>Microwave</b>                      | Yes  |
| <b>Basic UDI-DI No.</b>               | 57048311BU   |
| <b>UDI-DI HENROCUP® Clear with ml</b> | 05704831128777   |
| <b>UDI-DI HENROCUP® Green</b>         | 05704831128784   |
| <b>UDI-DI HENROCUP® Red</b>           | 05704831238791   |
| <b>UDI-DI HENROCUP Clear</b>          | 05704831128890   |
| <b>UDI-DI HENROCUP® Blue</b>          | 05704831128999   |
| <b>Standards and Regulations</b>      | Regulation (EC) 1935/2004 of 27. October 2004<br>Regulation (EC) 2023/2006 of 22 December 2006<br>Danish executive order, BEK nr 681 of 25/05/2020<br>Regulation (EC) 10/2011 of 14 January 2011<br>Regulation (EU) 2017/745 of 5 April 2017 |

| <b>Technical datasheet Lid</b>    |   |
|-----------------------------------|---|
| <b>Material Cup</b>               | LLDPE (Polyethylen)   |
| <b>Weight</b>                     | 13 Gr   |
| <b>Marked</b>                     | <b>CE</b>   |
| <b>Dishwasher</b>                 | <p>Dish washer 65° C</p> <p>80° C degrees Industry Dish Washer up to 3 minutes</p> <p>However, the lid can become dull and the quality deteriorates, so it is recommended to wash it by hand.</p>   |
| <b>Microwave</b>                  | No  |
| <b>Basic UDI-DI No.</b>           | 57048311BU  |
| <b>UDI-DI HENROCUP® lid small</b> | 5704831000271   |
| <b>UDI-DI HENROCUP lid Big</b>    | 5704831000011   |
| <b>Standards and Regulations</b>  | <p>Regulation (EC) 1935/2004 of 27. October 2004</p> <p>Regulation (EC) 2023/2006 of 22 December 2006</p> <p>Danish executive order, BEK nr 681 of 25/05/2020</p> <p>Regulation (EC) 10/2011 of 14 January 2011</p> <p>Regulation (EU) 2017/745 of 5 April 2017</p> |

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